

SB SALES



SECURA

Drain Tube & Chest Tube Securement Device



Product advantage

1. Replace the traditional skin fixation, reduce the patient dragging pain
2. Good absorption of the exudates around the drainage catheter, prevents the infection caused by the accumulation of seepage.
3. Transparent product structure makes it is easy for doctors and patients to view and control the drainage situation in time to avoid accidents.
4. Product structure is novel, safe, clean, beautiful and simple operation, easy for medical personnel to use.

Application Range

Suitable for fixation of different drainage catheters

Product Clinical Application

Model	Drainage catheter Size	Clinical Application
A	F-6-F12	Anesthesiology, and others
B	F-14-F18	ICU, Cardiothoracic surgery, hepatobiliary surgery, thoracic surgery, endoscopic surgery, breast, gynecology, gastroenterology, urology, etc
C	F-20-F24	ICU, Cardiothoracic surgery, hepatobiliary surgery, general surgery, urology, obstetrics and gynecology, etc
D	F-26-F30	ICU, Cardiothoracic surgery, hepatobiliary surgery, general surgery, urology, obstetrics and gynecology, gastrointestinal surgery etc
E	F-32-F36	ICU, Cardiothoracic surgery, hepatobiliary surgery, general surgery etc

Direction for use

1. Select the right size of the product in accordance to the drainage catheter size
2. Disinfect the skin by alcohol swab
3. Open the product package, take out the cotton pad, then put it on the catheter along the slit
4. Take out the lock and fixing plate, then put them on the catheter along the slit
5. Pinch the lock gently until the drainage catheter is clamped
6. Trap the drainage catheter by the fixed paste under the lock sets, remove the release paper opposite the fixed paste, then the drainage catheter can adhere to the skin surface

Fixed paste

Lock sets

Fixing plate

Sterile cotton pad



Single patient use



Read the user manual before use

STERILE

Sterilized by Eo

Do not resterilize

Unless opened or damaged, contents are sterile



SB SALES

521-522, Sec-8 IMT Manesar, Grugram, Haryana- 122051, India Tel : 0124-4203111 E: sbsalesind@gmail.com