




let's win together




# Pure Link Needleless Multi-dose Vial Adapter & IV Bag Access Spike


Vial Adapters & bag spikes are cost-effective solution for the safe and rapid transfer and reconstitution of drugs between vials, IV bags or IV bottles and syringes. The products are sterile devices which pierces the elastomeric septum of drugs contained in vials or IV bags. The products are effective way of prevention of needle stick injuries to the healthcare providers & also helps in prevention of infection.

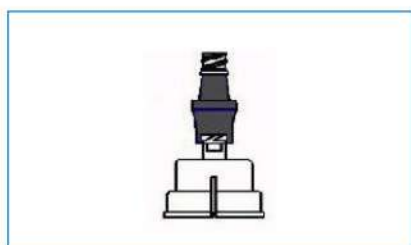

Multidose Vial Spike With Lid

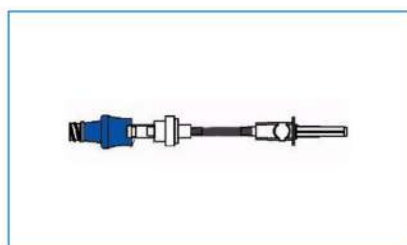

One Way Bag Spike

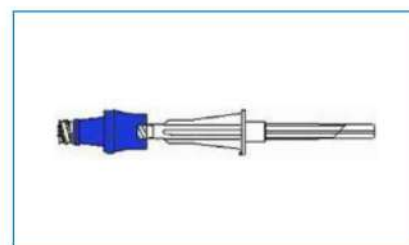

Two Way Bag Spike



Vial Spike




Multidose Vial Spike



Hydrophilic With Bacterial Filter

# Pure Link Needleless Multi-dose Vial Adapter & IV Bag Access Spike

## DIRECTIONS FOR USE

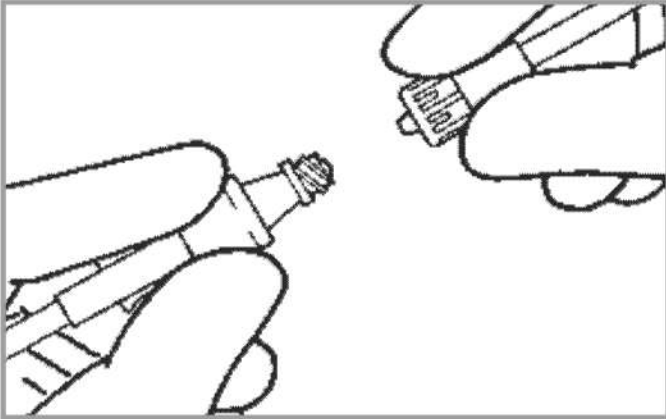


1. Peel open package at designated end and remove Pure Link Needleless Vial Adapter/ IV Bag Access Spike Do not contaminate adapter Spike.

2. Prepare drug vial according to manufacture's instructions.

3. Remove protective cover from adapter spike and center over vial stopper. Push straight down until spike penetrates prot and locks into place.

NOTE : use for one the connection to drug vial otherwise intended for needle access. Once connection is made DO NOT remove adapter form vial as it could cause damage to adapter or vial.

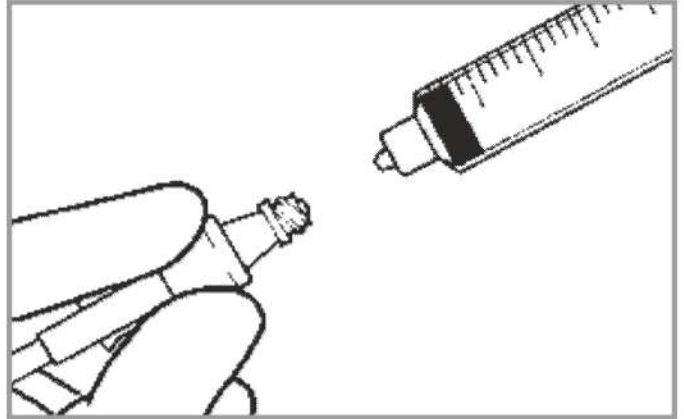


4. Swab silicone seal of Pure Link Needleless in accordance with facility protocol.

5. Attach syringe to Pure Link Needleless Push and twist until tight. Aspirate desired amount of fluid from vial according to facility protocol.

6. To disconnect syringe, grasp Pure Link Needleless and twist syringe away from Pure Link to pull apart.

7. For repeated access repeat from step four (4).

8. Dispose of Pure Link Needleless Vial Adapter with drug vial in accordance with facility protocol.

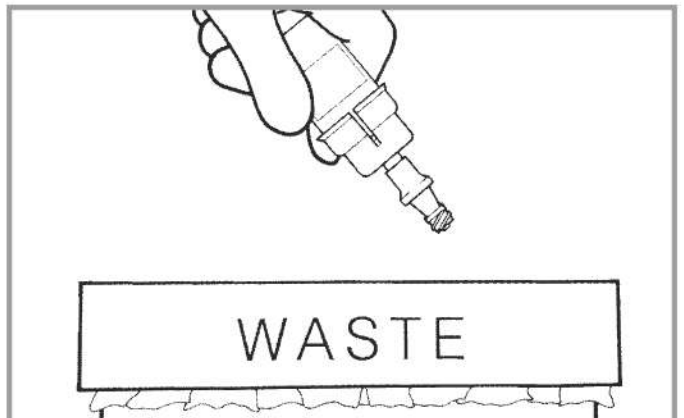

**SB Sales**

D-9, Jangpura Extension, New Delhi-110014 India , Tel : 91-0124-4147979 E-mail : sbsalesind@gmail.com