

SB SALES



STEADY-FIXTM

Endotracheal Tube Holder

Fits Endotracheal Tube Sizes 7.0-10.0mm

Covenient Checklist for Successful Application

- ✓ Prep the skin before applying the ET Tube Holder.
- ✓ Tightly spiral wrap strong adhesive tape around the channel and tube
- ✓ Be sure to wrap tape onto the tube at least 1" beyond the channel
- ✓ Support ventilator tubing



Steady-Fix Endotracheal Tube Holder




Application Instructions




1. Clean the E.T Tube application area. Make sure the area is dry before applying the ET Tube Holder. (a)




2. Wrap the E.T Tube Holder strap below the ears & around head of the patient. Secure the strap through the VELCRO tabs. (b)

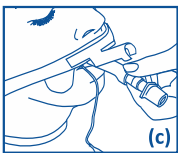


3. Fix the ET Tube into the flexible holder. (c)

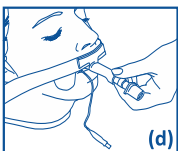

4. Clamp the ET Tube with the available removable clamp .(d)

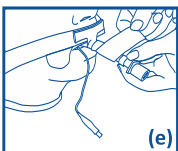

5. Remove the release cable tie from its retaining unit. Wrap it around the ET Tube & tighten it.

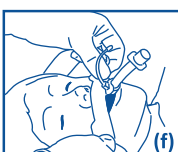

6. After checking the securement through the cable Tie cut the excess tab.

7. Periodically inspect the ET Tube Holder securement through the Adjustable clamp & cable Tie.

Tape Option: Fix the ET Tube inside the flexible holder then Use Strong Medical tape & wrap it to have a simple, safe & secure connections.(e)

Tube Repositioning: In order to reposition the Tube remove the cable Tie & adjustable clamp from the flexible holder. Reposition the ET Tube and again clamp & wrap the ET Tube with the cable Tie. If Medical Tape is being used just unwrap adjust & again wrap the holder & ET Tube together with the medical tape.

Removal: Unclamp the ET tube from the removable clamp & deflect cable release and push excess tab through retaining unit. In case of Medical Tape used to secure the ET Tube unwrap it.

Disposable: For single patient use not to be Re-Used.

For an instructional video about the product application visit www.sbshealthcare.com

Ordering Information

150010

Steady-Fix Endotracheal Tube Holder
Fits endotracheal tube sizes 7.0-10.0mm.
Individually packaged in box of 10.

Disposable. For single patient use.
Latex Free.



SB SALES

Head Office : D-9, Jangpura Extension,
New Delhi- 110014, India Tel:- 011-24325539
E-mail: sbsalesind@gmail.com

Factory : 521-522, Sec-8 IMT Manesar, Gurugram,
Haryana- 122051, India Tel: 0124-4203111, 08800783350
Web : www.sbshealthcare.com