



SecuraTM

Catheter Securement Devices

- Dependable
- Easy
- Secure
- Safe



CVC Securement




Foley Catheter Securement



Feeding Tube

The right securement

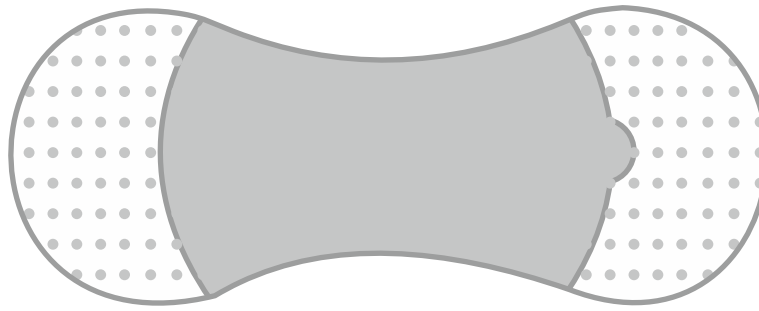

Secura devices are the most reliable and ideal choice for stabilizing Catherers and tubes. The product is strong enough to prevent dislodgment and / or migration, thus reducing the risk of lost lines, duplication of work, and patient discomfort of distress.

Features :

1. Hypallergenic, breathable and latex-free to reduce the risk of allergic reactions and skin irritation.
2. Strong adhesion makes sure that the lines are not to be dislodged by any binding.
3. Each plaster of Secura is placed in individual sterile unit to remove primary infection.
4. variety of product makes it applicable to different size of tubing and Catheters.
5. Easy to Apply, inspect and adjust the lines, tubes and Catheters.

Instructions to use :

1. Clean the skin and remove hair of required area according to the hospital protocol.
2. Take the product out of pouch open the fabric over-strap & remove inner release liner to expose adhesive.
3. Position the product under the line / hub place the line / hub in the center of the bottom adhesive strip.
4. Secure the top strap over the line / hub and press the adhesive in place. Pull and remove one side of the bottom release liner and secure the product to the skin. Then remove the other side of the bottom release liner and secure the product to the skin.
5. The top strap can be pulled back to inspect and adjust the line / tab.

Ordering Information

CODE	PACKAGING	SIZE	USAGE
SE 140010	100 Pcs / BOX - 20 Boxes / Carton	Small	for Lines and tubes sized 1.5-4.5 mm in dia
SE 140020	100 Pcs / BOX - 20 Boxes / Carton	Medium	for Lines and tubes sized 3-8mm in dia
SE 140030	100 Pcs / BOX - 20 Boxes / Carton		for cvc
SE 140040	100 Pcs / BOX - 10 Boxes / Carton	Large	for Lines and tubes sized 6-13mm in dia
SE 140050	100 Pcs / BOX - 10 Boxes / Carton	X Large	for Lines and tubes sized 6-13mm in dia



SB Sales

521-522, Sec-8 IMT Manesar, Grugram, Haryana- 122051, India
Tel : 0124-4203111 E: sbsalesind@gmail.com